




BI DESIGN

DIGITAL DENTAL CERAMICS


www.bidesignceramics.com

How to export STL files from 3shape Trios

1) At the 3shape scan page, chose file and confirm scan post processing is complete

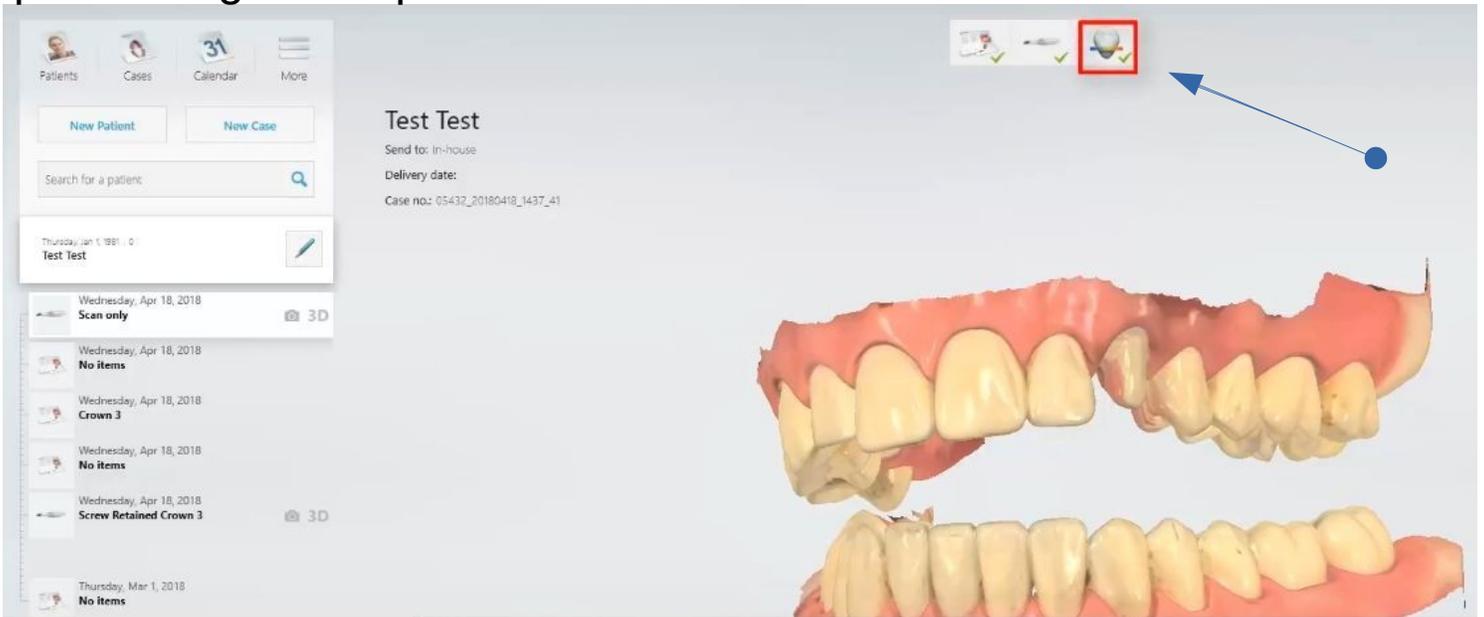


2) Right click on file for export, choose "Export", then "Scans"

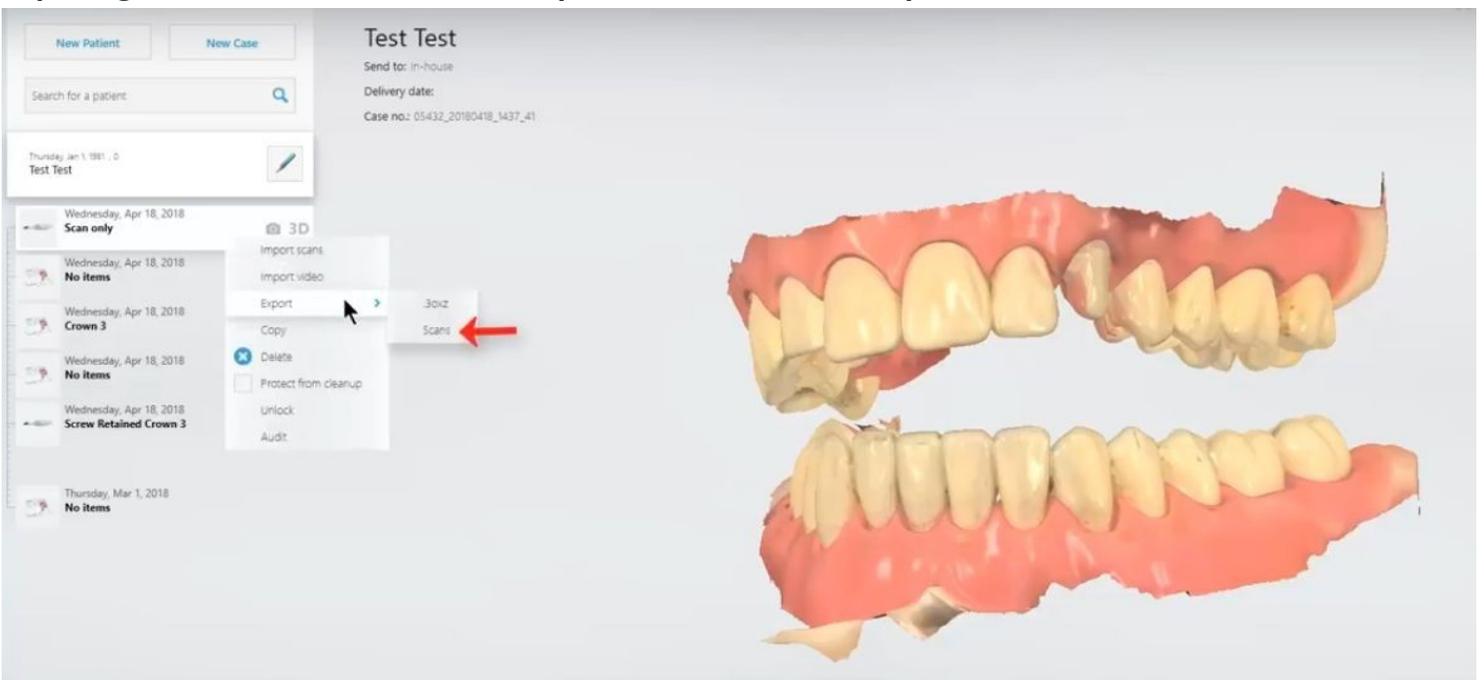

www.bidesignceramics.com

How to export STL files from 3shape Trios cont'd

3) In the save file window, click “Save as type” and choose “STL”

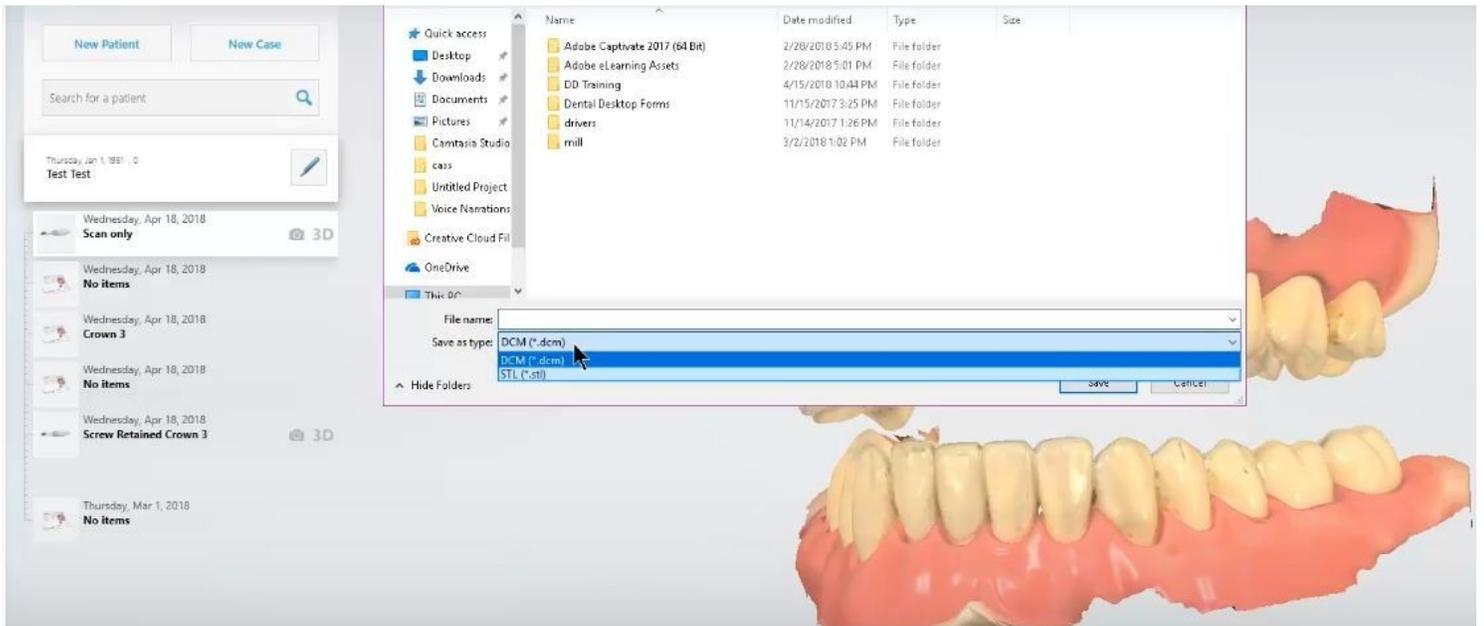

4) In “File name” enter “patient name and scan type”. Save in a folder titled with the “patient name” along with the additional patient scan files and a colour screen shot of the prep buccal side.

Now simply drag and drop or save the complete patient folder directly into your personal Drop box folder shared from “Bi Design Ceramics” for restoration processing.

Please email the patient lab script (yours or ours) to:
upload@bidesignceramics.com.

www.bidesignceramics.com